

Submittal Data Sheet

Features and Benefits

The Tri-Tech Med Touch Master Alarm Panel monitors and displays normal and alarm conditions from up to 128 remote medical gas source signals and provides alarm conditions as required by the latest edition of NFPA 99. The alarm is ETL listed to UL 1069 and CSA C22.2 # 205 Signal equipment. Pressure switches, transducers and DISS union check valve connectors are sold separately.

- **Five year parts and one year labor limited warranty***
- 2.85” LCD touch screen displays up to 8 normal and alarm conditions for pressure switch sensor inputs
- 2.85” LCD touch screen displays up to 8 normal and alarm conditions and provides a digital pressure display for up to 4 transducer sensor inputs
- 2.85” LCD touch screen displays normal and high/low alarm conditions for one 4 – 20 mA sensor input
- The LCD touch screen allows all alarm programming and set up to be done without the use of tools
- A green normal or red alarm condition for each gas service confirms the condition for each individual gas service
- Emergency preparedness instructions - Med Touch Alarm Panels allow users to set up customized instructions for each alarm signal which appear on the screen when the signal is in alarm
- Up to 128 remote master signals in a single panel
- Up to 4 master boards (32 signals) in a two slot, 8 (64 signals) in a three and 16 (128 signals) in a five slot panel
- Wireless (minimal low voltage wiring) models available
- Last event history (per gas board /signal point)
- Optional Ethernet board which provides; alarm notification to up to 5 addresses, website with real time graphic alarm face image, event log of last 100 events and Modbus interface with building management system.
- Optional BACnet module which provides BACnet interface of alarm conditions with building management system.
- Our Med Touch “T” Series of alarms may be interfaced with Lonworks using our 35-3032 Ethernet/ Rabbit board which provides a Modbus interface. Modbus provides interface with Lonworks.
- Made in the U.S.A.
- NFPA and ISO pre-loaded gas ‘labels’
- English, English/French and English/Spanish pre-loaded languages
- Editable text and alarm ‘labels’
- Self-contained unit - Designed for ease of installation and service
- Self-diagnostic error message display for ease of maintenance
- Audio and visual alarm indicators
- Bright easy to read LCD displays – clearly visible in both day and night lighting conditions
- Constant display and monitoring of each sensor input
- User programmable high/low set points on transducer and 4 – 20 mA sensor input boards
- Dry contacts provided on the CPU module for general fault condition covering entire panel

- Hinged frame for easy accessibility
- Color coded displays

* See Terms and Conditions, Document No. 99-0477, on our website at: www.tri-techmedical.com for complete details.




Additional features:

- Individual user programmable remote signal alarm points to accept NO or NC signals, or may be disabled. Factory preset to accept Normally Closed signals
- LCD indicators (Green) confirms normal status, (Red) indicates abnormal condition
- Menu of NFPA 99 master alarm signals for quick, easy selection
- Gas specific transducer master signal boards. An error message will be displayed if incorrect sensor or no sensor is attached
- Transducer master signal boards shall be capable of displaying gas readouts in PSI, kPa, BAR, inHg or mmHg
- Alarm repeat feature is factory set at 10 minutes, and is adjustable or may be turned off



Specification

The alarm shall be the Tri-Tech Medical Med Touch Master Alarm. The panel shall be microprocessor controlled and designed to comply with NFPA 99. The panel shall incorporate 2.85” LCD touch screen display(s) for groups of up to 8 signal points which shall allow all alarm programming and set up to be done without the use of tools. The unit shall be designed to accept an electrical input range of 120-240 volts AC – 50-60 hertz. The source voltage shall be stepped down with a self-contained transformer. The panel shall contain audible and visual alarm indicators. The audible alarm may be silenced by pressing the alarm silence button, but the visual alarm indicator can only be cancelled by fault correction. The alarm shall detect and filter out transient (less than 0.6 seconds) signals created by R.F.I. The alarm shall be capable of displaying alarm history for all possible alarm conditions. A security pin code shall be required and must be entered in order to access the programming software.

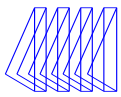

The following options shall be able to be added to the panel at the time of order or after installation;

- Optional Text / e mail notification of alarm events sent to up to 5 addresses
- Optional Ethernet module. This module may be added to any alarm panel. It will provide the following features:
 - Webserver – allows a remote user access to the alarm’s webpage – viewing a graphic image of the alarm with all signal conditions, pressure & event / history log information
 - Text notification of alarm conditions (up to 5 addresses)
 - Modbus and BACnet gateway interface
- Event / history log – maintains a rolling list of the 100 most recent alarm condition events and a file of the 1,000 most recent alarm events and is downloadable through embedded web pages
- Optional Ethernet connectivity with embedded web page
- Interface with building automation system via Ethernet
- Wireless (minimal low voltage wiring) models available

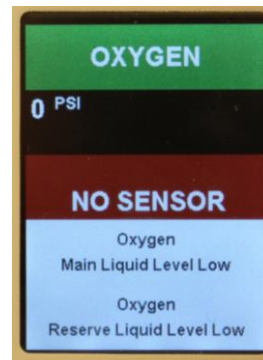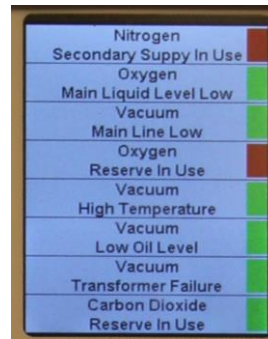


Master / Remote Signal Display (dry contact sensors)
 Shown in alarm condition displaying emergency instructions. Display blinks on and off when in alarm condition drawing attention and alternates displaying signal identification with user programmed emergency instructions.

Master / Remote Signal Display (both dry and transducer sensors)
 Shown in normal condition. User selects signal labels from database of NFPA signals included in the boards memory. Individual signal points are pre-programmed NC but may be re-programmed as Disabled or NO by user.

Master / Remote Signal Display (dry contact sensors)
 Shown in alarm condition. Display blinks on and off when in alarm condition drawing attention and alternates displaying signal identification with user programmed emergency instructions.

Master / Remote Signal Display (transducer sensors)
 Shown in alarm condition. The standard display looks and acts the same as both of the above Master/Remote displays. By touching the signal point you wish to view, the display changes to the view shown here providing the additional actual line pressure of the gas being monitored. This may be viewed at any time, in Normal or Alarm condition.

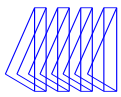

How to Order: Easy to use modular ordering system.

Fill in the blanks to specify the Med Touch Master Alarm that meets **your** needs.

Model Numbering:

Signal Number Group

			001 008	009 016	017 024	025 032	033 040	041 048	049 056	057 064	065 072	073 080	081 088	089 096	097 104	105 112	113 120	121 128
T	N	C	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S

2 Slot Front

3 Slot Front

5 Slot Front

Note: a 35-3032 kit may be ordered separately and installed in the field to provide; Ethernet webserver, event log, alarm notifications may be sent up to 5 users via text or e-mail and Modbus Interface. The 35-3033 Ethernet programming kit is required and must be ordered when the 35-3032 kit is ordered. Only one is needed per facility.

Note: a 35-3035 kit may be ordered separately and installed in the field to provide a BACnet interface. The 35-3032 and 35-3033 kits are also required whenever the 35-3035 kits is ordered. Only one 35-3033 kit is needed per facility.

S = Master Signal Input Type

Non-Latching Alarm Relays	Description
M	4-20mA
P	Dry Contact (8 Signal Points)
Q	Transducer (4 Gases, 8 Signal Points)

Latching Alarm Relays	Description
%	4-20mA
+	Dry Contact (8 Signal Points)
#	Transducer (4 Gases, 8 Signal Points)

B – Blank Slot - 4 Vertical Future Expansion (Blank) Positions in One slot

F – Future Expansion (Blank) Position

List Signal Input Type Choices from Left to Right

Starting at Signal Number Group 001 thru 008

C = Gas Set

U – Tri-Tech Labeled **NFPA** Color Code English

I – Tri-Tech Labeled **ISO** Color Code English/French

S – Tri-Tech Labeled **ISO** Color Code English/Spanish

N – Tri-Tech Labeled **NFPA** Color Code Spanish

N = Number of Slots on Front Panel


2 – 2 Slot Front Panel

3 – 3 Slot Front Panel


5 – 5 Slot Front Panel

T = Type of Alarm


T - Touch Screen Digital

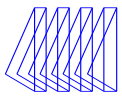


Part No. Examples

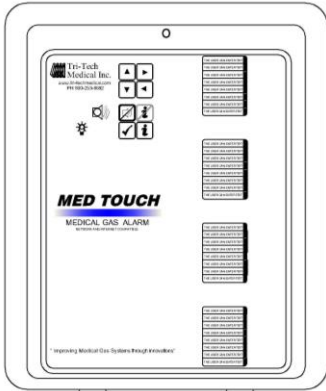


T2UPPPP = 2 Slot Master Alarm Tri-Tech labeled, set for NFPA Color Code English. 32 Dry Contact Signal Points w/ Non-Latching Alarm Relays. No Interface Communication Option

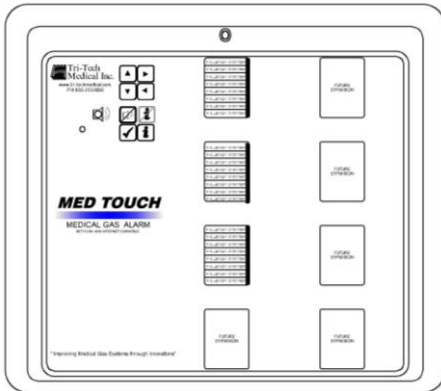

T3I+++FB = 3 Slot Master Alarm Tri-Tech labeled, set for ISO Color Code English/French gas set. 24 Dry Contact Signal Points w/ Latching Alarm Relays and one Future Expansion (Blank) position at bottom of 2nd slot, 3rd slot Blank. No Interface Communication Option.

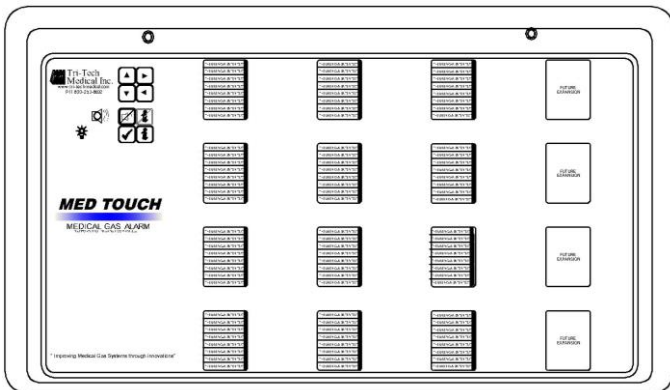

T5UPPPPPPPQQQB3 = 5 Slot Master Alarm Tri-Tech labeled, set for NFPA Color Code English gas set. 64 Dry Contact Signal Points w/ Non-Latching Alarm Relays and 32 Transducer Signal Points W/ Non-Latching Alarm Relays, 5th slot is Blank. Ethernet Web Server, Modbus, and BACnet Interface Communication Option.

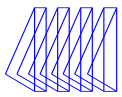

DIMENSIONS - INCHES
(MM)

